



READMATE LED DESKTOP MAGNIFIER


- 1** Open battery compartment on the bottom side of the Magnifier and remove plastic insert from it.

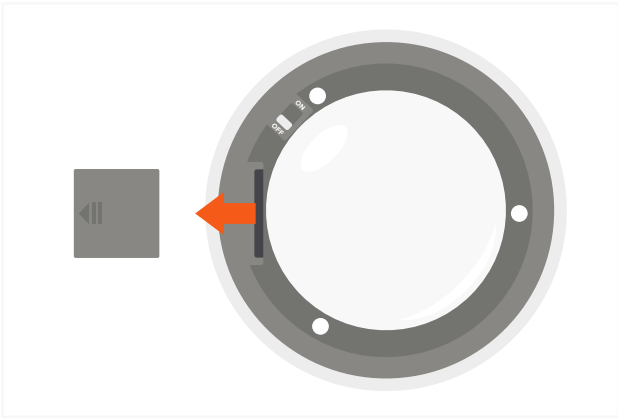


- 2** Place 2 lithium ion batteries in battery case and snap the battery cover back in place.

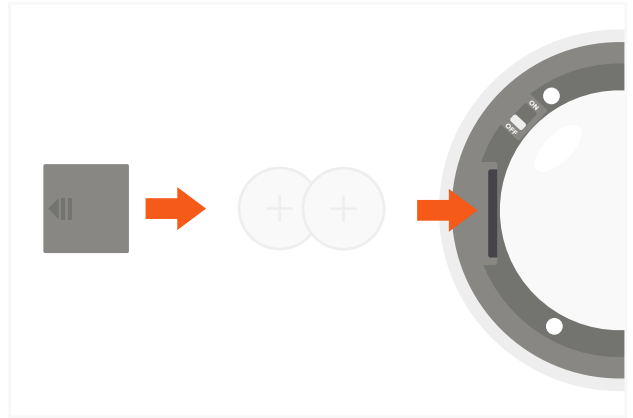

- 3** Put On/Off switch on the bottom side of the Magnifier into position "ON".

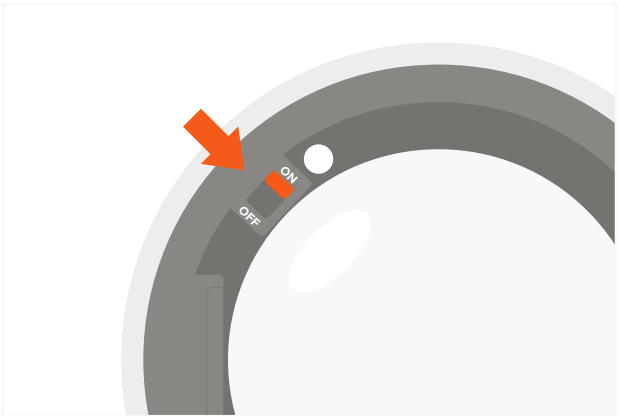

- 4** To turn on the LED lights, simply press down on the LED Magnifier. Press again to turn off.

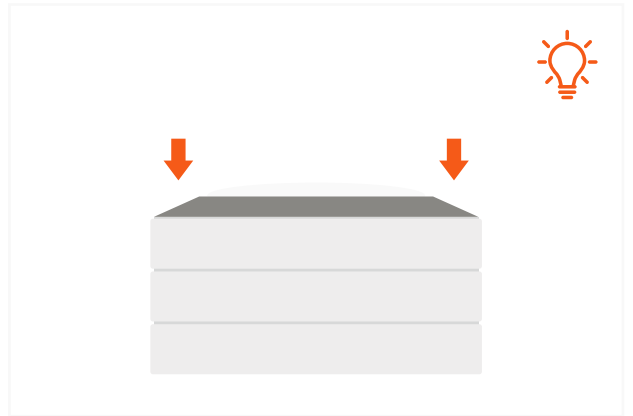

- 5** To magnify text or images move the magnifier across a flat surface with one hand.



- 6** For close observation hold the Magnifier up to an object.

