




## Quick Start Guide

For use with DryCaddy UV




**EAR TECHNOLOGY CORPORATION** Helping people hear better, every day.

### 1 Installing the Batteries

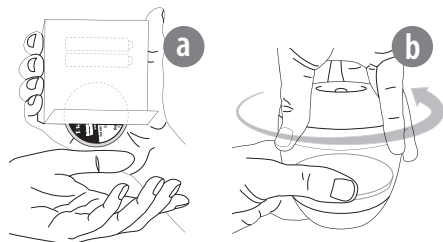

One DryCaddy Disc desiccant and two AAA batteries are packaged in the small inner box. Remove the contents.


Remove lid / battery cover by turning counterclockwise.

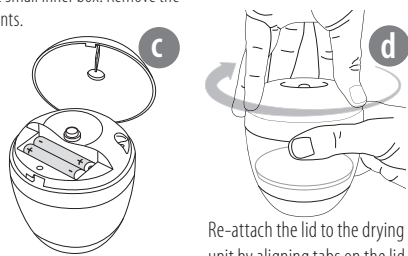

Install batteries with correct polarity.


Re-attach the lid to the drying unit by aligning tabs on the lid with notches on the unit. Turn clockwise to close.

### 2 Installing the Desiccant

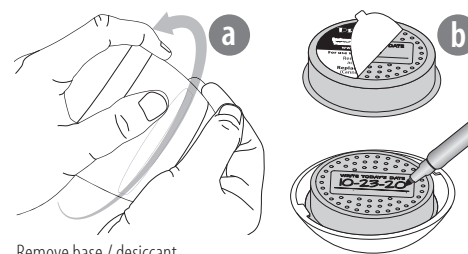


Remove base / desiccant compartment by turning counter-clockwise.

Remove foil from DryCaddy Disc desiccant. Place in desiccant compartment, holes up. Write today's date on the desiccant as a reminder to replace in one month.

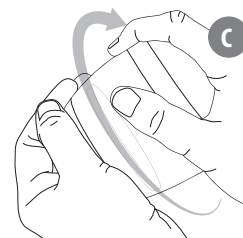


Re-attach the base to the drying unit by aligning tabs on the base with notches on the unit. Turn clockwise to close.

### 3 Adding Hearing Devices

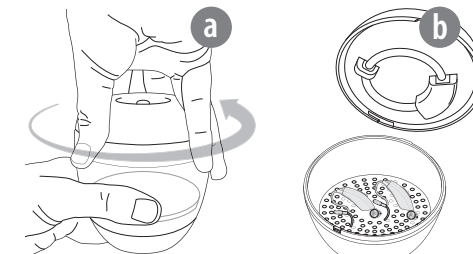

Remove lid and battery compartment sections by turning counterclockwise.

Add hearing devices to the mirrored surface. Be sure to open the battery door. You can leave the batteries inside.

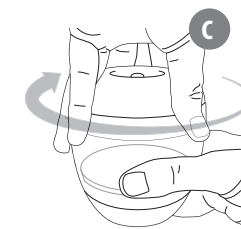


Re-attach the lid to the drying unit by aligning tabs on the lid with notches on the unit. Turn clockwise to close.


### What's included



### MAIN COMPONENTS OF DRYER

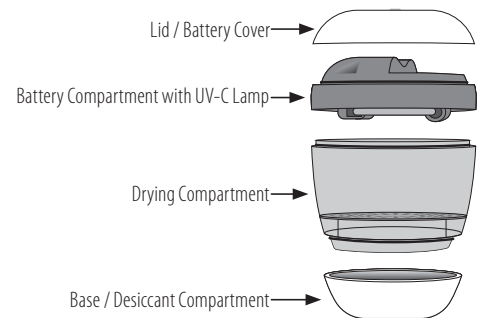


### 4 Activating the UV-C Cycle

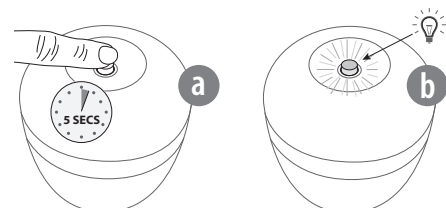

Hold button for 5 seconds then release.

Red light will flash.

**NOTE:** The UV-C lamp may not start immediately if it has not been used for long periods of time. If the lamp does not turn on, please restart the unit.

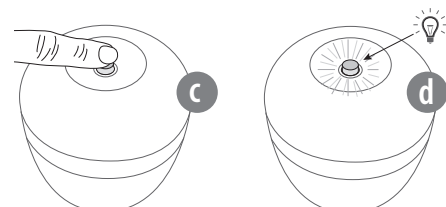


Press and release button once more. This starts the UV-C function.

Light will slowly pulse for approximately 3-1/2 minutes then shut off automatically. The UV-C function is complete.

When not in use, your devices should be kept in the DryCaddy UV appliance with the lid closed. The desiccant will continue to remove moisture from the devices.

### Ending the Cycle Manually

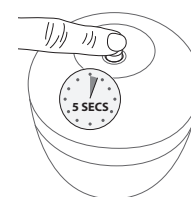

While the UV-C lamp shuts off automatically in 3-1/2 minutes, you may want to stop the cycle before it is complete. To do so, simply press and hold the LED power light for 5 seconds. The red light will go off indicating the cycle has stopped.

### IMPORTANT SAFETY INFORMATION

This product is for household use. When using products powered by batteries, especially when children are present, basic safety precautions should always be followed, including:

**DANGER**—To reduce the risk of electrocution:

- Do not use while bathing.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not place in or drop into water or other liquid.
- Do not reach for this device if it has fallen into water.

**WARNING**—To reduce the risk of burns, electrocution, fire, or injury to persons:

- **NEVER open the drying compartment while the UV light is on. Looking at the UV light with the naked eye may cause severe burns to the eye.**
- Closely supervise use by, on or near children or invalids.
- Batteries should be kept out of reach of children.
- Use this product only for its intended use as described in this guide.
- Never operate this product if it is not working properly, if it has been dropped or damaged, or if the batteries have corroded. Return it to the place of purchase or service center for examination and repair.
- Never store batteries in a loose container where they can risk touching other items.
- Never place the product on a soft surface such as a bed. Keep any air openings free of lint, hair, and the like.

**NOTE:** This product is not designed to diagnose, prevent, monitor, treat, or alleviate disease.

**OPERATING CONDITIONS** The operating temperature of this appliance is approximately ambient temperature.