

RingConn

English


1 Download the RingConn App:

Search for "RingConn" in the App Store or Android application stores, or scan the QR code to download, and follow the prompts to register an account.



2 Activate the ring for the first time:

Put the ring into the Charging Dock. Blue blinking of the indicator light means that the ring is activated, and is ready for being bound to a cell phone.

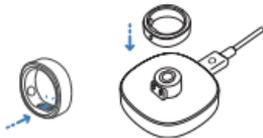

3 Connect the ring with the App:

Open the RingConn App, enter the [Me] page, select the binding device, and follow the prompts.


- Turn on the Bluetooth of the cell phone, and keep the ring within one meter from the cell phone.



4 Wear the ring properly:

Turn the ring to make it fit the finger, and keep wearing it properly.


- Adjust the position in time if it is shifted.
- Before using it for the first time, it is best to charge it for 60 minutes or make sure its battery level is at least 80%.

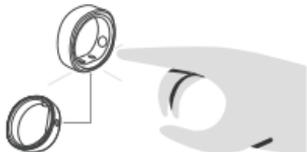

5 View data:

Synchronize and view health data and power information through the RingConn App.

- It is recommended to wear it for a long time and synchronize the data every day, pay attention to the battery indicator, and charge when needed.

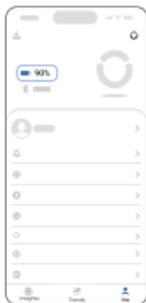

6 Indicator light status:


Ring

-  Blinking blue: the ring is activated and enters pairing mode
-  Solid blue for 3 seconds: the ring exits flight mode



Charging Dock

-  Solid white for 3 seconds:
The charging dock is turned on and ready for charging the ring
-  Blinking white: The ring is charging
-  Solid white: The ring is fully charged

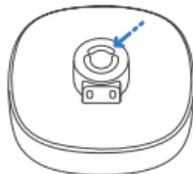



Version:
V2.0/RCA-02



(No.2845)



(No.1942)



RoHS



OTHER

Made in China

中国制造