


# General Use Face Mask

## USER INSTRUCTIONS

Thank you for your purchase!

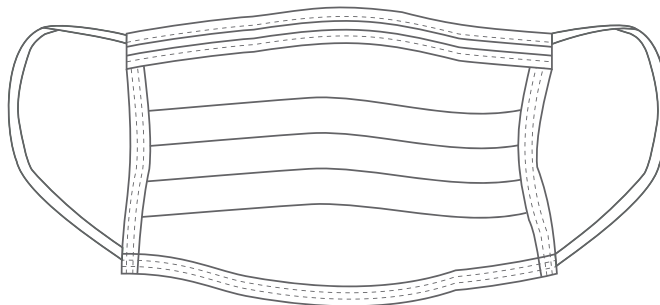

**NOT FOR MEDICAL USE.**  
**INTENDED FOR SINGLE USE ONLY – DISCARD AFTER USE**

**FLTR, Inc**  
**[www.FLTR.com](http://www.FLTR.com)**  
Toll free: 1-888-595-3587  
(Monday - Friday 9am - 4:30pm PST)

### FEATURES:


- ≥95% Bacterial Filtration
- 3-Layer Construction for Protection and Comfort
- Elastic Ear Loop Design

### PROPER FIT INSTRUCTIONS:


For proper use, follow these steps each time face mask is worn.

1. Elastic straps should be pre-stretched before placing face mask on the face (Fig. 1).
2. Cup the face mask in your hand, with the face mask cupped in your hand, allowing elastic straps to hang freely (Fig. 2).
3. Align face mask under chin with the nose covering area facing up. Pull each strap over your ears, allowing it to rest behind each ear. (Fig. 3).
4. Use both hands to align nose covering area over nose and adjust the to fit the shape of your face comfortably (Fig. 4). ALERT! Not properly securing the nosepiece by only using one hand may result in improper fit, decreased filtration, and face mask contamination. Use two hands to secure nosepiece.
5. Make sure the mask is completely secure. Make sure it covers your nose and mouth so that the bottom edge is under your chin. (Fig. 5).

### FIT INSTRUCTIONS:

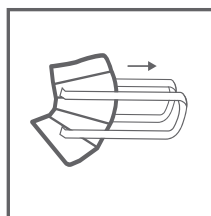

(Fig. 1)

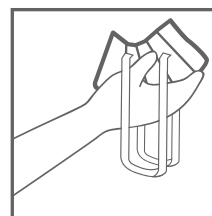

(Fig. 2)



(Fig. 3)



(Fig. 4)



(Fig. 5)