snūza®

HERO

Portable Baby Movement Monitor

07/2015

ENGLISH

Instruction Manual

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ESPAÑOL

NEDERLANDS

SNUZA is a registered trademark of
Biosentronics 

Made in South Africa
 Patented


CONTENTS

1	IMPORTANT – READ FIRST!	4
2.	Introduction	6
3.	Parts Identification	7
4.	Quick Start Guide	8
5.	Diaper (Nappy) Fitment	8
6.	Fitting your Hero	9
7.	Operation	9
7.1	Startup	9
7.2	Monitoring	10
7.3	Travelling	11
7.4	Clothing	11
7.5	Baby Sound and Video Monitors	11
8.	Settings	12
8.1	Audible Tick	12
8.2	Rouse Warning	12
8.3	Movement-Rate Warning	13
9.	Battery	13
9.1	Battery Indicator	13
9.2	Battery Life	13
9.3	Replacing the Battery	14
9.4	Disposal	15
10.	Maintenance	15
10.1	Precautions	15
10.2	Cleaning and Storage	15
11.	Troubleshooting	15
12.	Disclaimer	16
13.	Guarantee	16

1. IMPORTANT - READ FIRST!

It is essential that Hero's coloured flexible tip maintains constant contact with baby's tummy.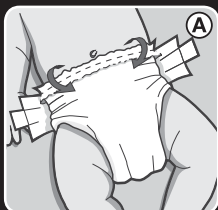



ALWAYS:

Use the correct size diaper (nappy). If it is too loose, Hero may lose contact.

Fold the top of the diaper over before attaching Hero, to ensure a firm grip. Use the diaper adjusters to provide a snug fit around the tummy.


Firmly clip Hero onto the folded diaper waistband, using the full extent of the clip.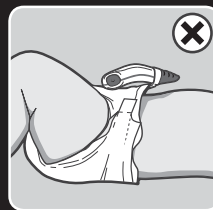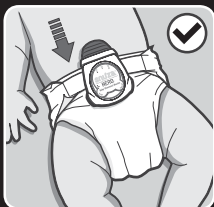

Ensure that Hero sits squarely on the diaper, and that the Movement Sensor (1) is in contact with baby's tummy.



Press the ON/OFF button (6) to start monitoring.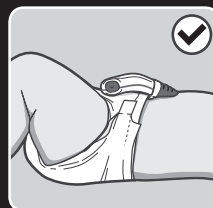

Always follow the above procedure to ensure that your baby's abdominal movement is being monitored, and to prevent false alarms caused by lack of contact with the baby.

NOTE:

Handle Hero with care to prevent damage to clip or sensor.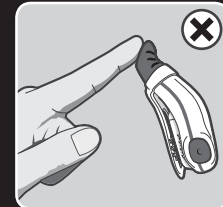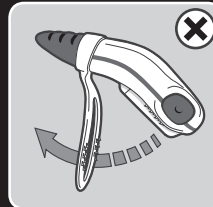

2. INTRODUCTION

Hero is a portable baby movement monitor which safely and conveniently monitors your baby's abdominal movements while sleeping.

Hero can be clipped onto your baby's diaper (nappy) for total mobility. No cords, wires, sensor pads or external power are required.

Clip on, switch on and relax.

Hero detects even the slightest abdominal movement and will alert you if your baby's abdominal movements are very weak or fall to less than 8 movements per minute. If no abdominal movement at all is detected for a period of 15 seconds, Hero will vibrate gently. Often this vibration is enough to rouse the baby, and Hero will revert to monitoring mode.

After three vibration/rouse incidents, the Rouse Warning will alert you to the fact that your baby's abdominal movements have stopped for 15 seconds on three occasions.

If no further abdominal movement is detected for another 5 seconds, an alarm will sound to alert you.

Hero is not suitable for co-sleeping babies or bed sharing.


3. PARTS IDENTIFICATION

Front

Reverse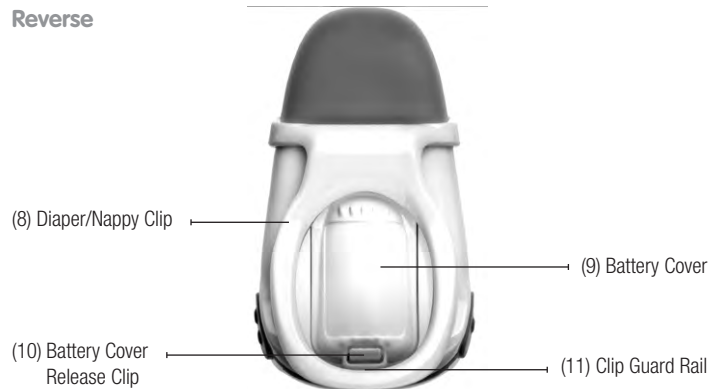


4. QUICK START GUIDE

- Important:** Use the correct diaper size and make sure it fits snugly (see Section 5 *Diaper/Nappy Fitment*).
- Switch on by briefly pressing the ON/OFF button (6).
- A start-up test will illuminate the indicator lights, vibrate the stimulator, play the start-up tone and perform a battery test.
- Clip Hero onto the diaper so that the Movement Sensor (1) is in direct contact with the baby's tummy (see Section 6 *Fitting your Hero*).
- The Movement Indicator Light (3) flashes on each abdominal movement. See Section 7.2 *Monitoring* for a full description of what it signifies.
- If abdominal movement is not detected for 15 seconds, the vibrating stimulator will attempt to rouse the baby.
- If further abdominal movement is detected within 5 seconds after a vibration, the Movement Indicator Light (3) will flash red to indicate that there has been a 15-second period of no abdominal movement.
- After three vibrations, the Rouse Warning is activated – Hero beeps every 2 seconds and the Movement Indicator Light (3) flashes red with every beep.
- If the period of no abdominal movement reaches 20 seconds, the full audible alarm will sound and the Alarm Indicator Light (4) will flash red.
- The alarm can be deactivated by briefly pressing the MODE/OFF button (7) and monitoring will continue.
- To switch off the monitor, press both the ON/OFF (6) and MODE/OFF (7) buttons simultaneously. A shutdown tone will be heard.

5. DIAPER/NAPPY FITMENT


Important: Correct diaper fitment is essential for the operation of Hero. The thin flap of material at the top of the diaper must be folded over the thicker part before the diaper is closed (this is the section to which Hero will be attached). The diaper should fit snugly on the baby with the top of the diaper just below the belly button. Loose fitting diapers will gape, causing the monitor to lose contact with the baby and false alarm. A very 'full' or wet diaper may also cause it to gape.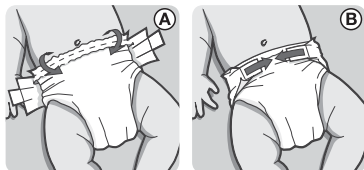

6. FITTING YOUR SNUZA

When attaching Hero to the diaper, put your fingers between the diaper and baby's skin in order to make sure that you do not trap baby's skin in the clip.

Ensure that the Battery Cover (9) is secure. Clip your Hero *all the way* onto the diaper to the full extent of the clip, as shown: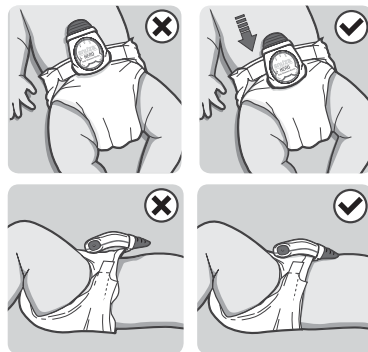

Make sure that the Movement Sensor (1) is in contact with the baby's tummy around the belly button area, although closer to the sides may suit different sleeping positions.

* The soft sensor tip on the monitor is made from medical grade silicone and is hypoallergenic. In the unlikely event that a red friction mark is noticed on the baby's skin where the monitor was positioned, then we recommend that a thin layer of clothing is placed between the baby's skin and the monitor. The position of the monitor must also be changed to a different location on the tummy, away from where the red mark was noticed and preferably closer to the side.

The World Health Organisation (WHO) recommends that babies be placed to sleep on their back unless advised otherwise by a doctor.

7. OPERATION

7.1 Startup

Switch on by briefly pressing the ON/OFF button (6).

A startup-test will run which will turn on (in sequence) all indicator lights, vibration stimulator and audible alarm. The Battery Indicator Light (2) will flash briefly as follows:

- | | | |
|--|---------------|--|
| | Green | Battery OK |
| | Orange | Battery nearing depletion – replace battery |
| | Red | Battery flat – Hero will not monitor, replace battery |

Warning: Ensure that all lights, vibrator and alarm are working properly and that the battery level is high enough for proper operation (i.e. green or orange light). **Otherwise, DO NOT use the monitor.** If the battery is too flat to operate, the Battery Indicator Light (2) will flash red accompanied by a short beep every second. **Change the battery before further use** (see Section 9.3 *Replacing the Battery*).

To switch OFF, press both the ON/OFF button (6) and the MODE/OFF button (7) simultaneously.

7.2 Monitoring

During monitoring, the Movement Indicator Light (3) provides information about your baby's abdominal movements, while the Alarm Indicator Light (4) alerts you to potential problems.

The Movement Indicator Light (3) flashes with each abdominal movement:

- Green Normal abdominal movements.
- Orange Very small or weak abdominal movements (eg very shallow breathing).
- 🎵 Orange and Movement rate Warning sounds (dual-tone beep every second) Movement-rate below 8 abdominal movements per minute. If the abdominal movement rate increases to over 8 movements per minute, the Movement Indicator Light will reset to green (normal) and the warning will stop. To disable this function, see Section 8.3 *Movement-Rate Warning*.
- Red There has been at least one incident of no abdominal movement for 15 seconds, where abdominal movement was again detected after the vibration (see below).
- 🎵 Red and Rouse Warning sounds (beep every 2 seconds) The Rouse Warning indicates that there have been 3 vibration/rousing incidents. To disable this function, see Section 8.2 *Rouse Warning*.

The Alarm Indicator Light (4) signifies the following:

- Flashes (1) red with vibrating stimulus If no abdominal movement at all is detected for 15 seconds, Hero gently vibrates in an attempt to rouse the baby. The Alarm Indicator Light (4) flashes red simultaneously. If abdominal movement is detected again within the next 5 seconds, the Movement Indicator Light (3) will flash red on each subsequent abdominal movement. This is an indication that there has been an incident of no abdominal movement for 15 seconds.
- Flashes (4) red and alarm sounds If no abdominal movement at all is detected for a further 5 seconds after the vibration stimulus (20 seconds in total), the Alarm Indicator Light (4) flashes red and the full audible alarm sounds. The alarm can be deactivated by briefly pressing the MODE/OFF button (7) or by turning Hero off completely (press ON/OFF (6) and MODE/OFF (7) buttons simultaneously). If the MODE/OFF button is used to deactivate the alarm, monitoring will continue.

7.3 Travelling

Movement from external sources such as being pushed in a pram or travelling in a vehicle or aircraft may be detected by Hero's sensor. *Hero should not be relied upon where external sources of movement are present.*

7.4 Clothing

- Hero can be clipped onto the folded waistband of the diaper or the waistband of elasticated pants, as long as the Movement Sensor (1) touches the baby in the tummy area.
- There may also be a thin layer of clothing (eg a vest) between the Sensor and the skin, but *ensure that the Sensor maintains contact*.
- The monitor can also be used if baby is dressed in light clothing (for example, a Babygro) over the nappy or pants to which the Hero is attached.
- Too much clothing may attenuate or muffle the alarm. Make sure the alarm can be heard if baby is covered in clothing and/or blankets.

7.5 Baby Sound and Video Monitors (Optional)

Hero can be used together with a baby sound or video monitor to extend the distance at which the alarm can be heard.

8. SETTINGS

8.1 Audible Tick

This feature causes Hero to emit an audible 'tick' on each abdominal movement. To enable the Audible tick, ensure that Hero is switched ON, and press the MODE/OFF button (7) briefly (less than 1 second). To increase the volume of the tick, press the MODE/OFF button once again for medium volume and again for high volume.

To turn off, briefly press the MODE/OFF button a fourth time – Hero will give a louder beep.

This feature may be used with a baby sound monitor to enable you to 'hear' baby's movements from a distance.

8.2 Rouse Warning

During monitoring, if no abdominal movement is detected for 15 seconds, Hero vibrates gently in an attempt to rouse the baby. If abdominal movement resumes within the next 5 seconds, the Movement Indicator Light (3) will flash red on each subsequent abdominal movement as an indication that there has been an incident of no movement for 15 seconds.

With the Rouse Warning enabled, after 3 such vibration/rousing incidents, the Movement Indicator Light (3) will change to flash red *every 2 seconds* and Hero will emit an audible beep every 2 seconds. This alerts you to the fact that your baby has stopped all abdominal movement for 15 seconds on three occasions and has then resumed abdominal movement after Hero has vibrated.

Note:- If abdominal movement does not resume within 5 seconds after a vibration, the full audible alarm will sound and the Alarm Indicator Light (4) will flash red. The Rouse Warning is automatically enabled each time Hero is switched ON. To disable this warning, ensure that Hero is switched ON, and press and hold the MODE/OFF button (7) for at least 1 second until a tone is heard and the unit vibrates briefly.

If you want to enable the warning again, switch Hero off completely (press both ON/OFF (6) and MODE/OFF (7) buttons simultaneously), then switch it ON again (the rouse warning is enabled by default when Hero is switched on).

8.3 Movement-Rate Warning

The Movement-rate Warning alerts you if the baby moves less than 8 times per minute – the Movement Indicator Light (3) flashes orange and the Movement-rate Warning sounds (dual-tone beep every second). This feature is automatically enabled each time Hero is switched ON.

To disable this warning during monitoring, ensure that Hero is switched ON, and press and hold the ON/OFF button (6) for more than 1 second until the unit beeps.

If you want to enable the warning again, switch Hero off completely (press both ON/OFF (6) and MODE/OFF (7) buttons simultaneously), then switch it ON again. The Movement-rate Warning is enabled by default when Hero is switched on.

9. BATTERY

9.1 Battery Indicator

If the battery becomes depleted during monitoring, the Battery Indicator Light (2) will flash as follows:

- | | | |
|---|--|--|
|  | Red flash on each abdominal movement | Nearing depletion – replace battery before further use. |
|  | Red flash on each abdominal movement and the low battery alarm beeps every 2 seconds | Battery flat – No monitoring will take place. Replace battery before further use. |

If the battery is nearing depletion or depleted, change the battery before further use of the monitor (see Section 9.3 *Replacing the Battery*).

9.2 Battery Life

The life of the replaceable battery depends entirely on usage. Battery life will vary significantly depending on the average daily duration of use and/or alarm duration and/or number of alarms. Turning on and off unnecessarily shortens the battery life. If the number of alarms is high, check diaper fitment.

Please note that the product guarantee does not apply to the battery.




9.3 Replacing the Battery

For all SE models: use a CR2 (CR15270) battery available from most retail outlets.

For all other models: use only a 14250 battery that delivers 3V 600mAh, available from a specialist battery supplier or your national Snuza distributor (please see our website www.snuza.com for information about batteries and details of your national distributor).

**** The serial number is located inside the battery housing (it can also be found on the original Snuza packaging). To locate the serial number, remove the battery cover and battery as per the instructions below:**

1. Use your fingernail or a small flat implement. With a pinching action (Figure 1), gently pull back the Battery Cover Release Clip (10) on the underside of the monitor and lift the Battery Cover at the same time (Figure 2).
2. The Clip Guard Rail (11) prevents the clip from bending too far. Do not try to bend the clip beyond this point.
3. Remove the depleted battery. **The serial number is located beneath the battery.**
4. Insert the new battery, negative end first, as shown in Figure 3. Ensure that the polarity is correct (align the positive end '+' of the battery with the '+' symbol in the battery housing). Replace the Battery Cover and ensure that it is correctly closed (Figure 4). **Do not use the monitor without the Battery Cover in place.**
5. Test the unit by switching on and ensuring that the Battery Indicator Light (2) flashes green.
6. If the unit starts emitting a beep after the battery has been replaced, simply press the ON/OFF and MODE/OFF buttons simultaneously to turn it off.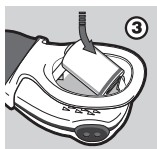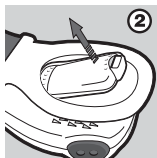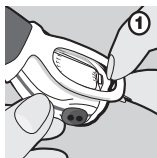



9.4 Disposal

The battery must be disposed of in accordance with your country's laws governing the disposal and recycling of batteries.

10. MAINTENANCE

10.1 Precautions

- Do not insert anything into the Speaker Holes (5).
- Do not allow any liquid to enter the Speaker Holes.
- Avoid bending the sensor unnecessarily (fig 1).
- Do not use undue force to bend the clip (fig 2).
- Do not pull the Battery Cover Release Clip (10) beyond the Guard Rail (11).

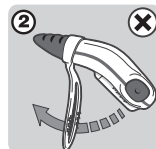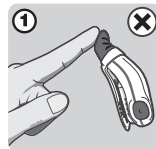10.2 Cleaning and Storage

- Gently wipe with a damp cloth to remove any dirt. A mild soap may be used for stubborn dirt.
- Do not use any other cleaning agents as they may contain damaging chemicals.
- The monitor is splash resistant, not waterproof. Do not immerse in water. If it gets wet, dry it off immediately.
- When changing the battery, dirt that is trapped between the battery replacement hatch and the case may be cleaned with a dry cotton bud once the Battery Cover (9) has been removed. Be sure that the rubber seal is free of dirt before replacing the Battery Cover, as this could compromise the seal.
- When not in use, keep your Hero in its protective case. Do not store in a damp area.

11. TROUBLESHOOTING

1. Check incorrect fitment and placing of Hero on baby's diaper (see Section 5 *Diaper/ Nappy Fitment* and Section 6 *Fitting your Hero*).
2. Battery failure. The Battery Indicator Light (2) will show red (see Section 9.3 *Replacing the Battery*).
Please visit our website www.snuza.com for further information.

12. DISCLAIMER

Customers use this monitor at their own risk and Biosentronics cc, its members, employees, agents, contractors, representatives, manufacturers, distributors and associates ("SNUZA") shall not be held liable for any loss, damage or injury suffered by the customer or any other person arising from, or in connection with, the use or manufacture of the monitor, including loss, damage or injury suffered by the customer or any other person as a result of any defects in respect of the monitor, and all warranties, and any other liability from whatsoever cause arising, are hereby excluded and the customer hereby indemnifies Biosentronics cc in respect of any such liability.

13. GUARANTEE

Biosentronics cc guarantees this product to be free from defects in materials and workmanship for a period of 12 months from the original date of purchase, verified by a sales receipt. Our guarantee liability shall be limited to replacing the unit free of charge and does not include transport costs. Our guarantee extends to all countries in which this product is supplied. By the act of operating this product, the user accepts all resulting liability. We reserve the right to modify the provisions of this guarantee at any time without notice. This guarantee does not cover the battery, incorrect application, tampering, misuse or transport damage.

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