




EXPLORE LI +

QUICK START GUIDE

START HERE

STEP 1: Charge Before Use

We recommend fully charging your hearing aids and the charger for a duration of 2 hours prior to first use.

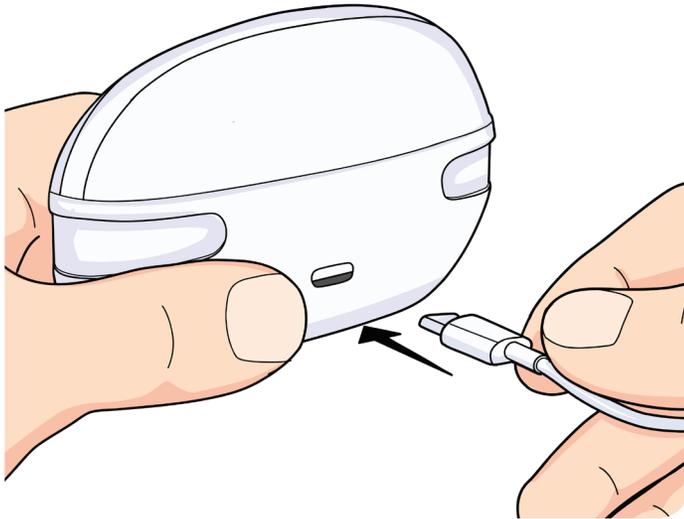

STEP 2: Download the HA FIT APP

The HA FIT APP lets you easily fit your hearing aids yourself and adjust the sound to suit your hearing loss.

Download the HA FIT APP prior to using Explore Li+
Scan the QR code below to download the APP.



Quick Bluetooth Setup

Watch our short video to easily connect your Explore Li+ hearing aids, pair with the app, and start streaming in minutes.

Scan the QR Code below or by accessing:




<https://www.earahearing.com/pages/5-0-bluetooth-streaming>

STEP 3: Remove Hearing Aids From Charger

Once your hearing aids are fully charged, simply remove them from the charger.

Explore Li+ will turn on automatically.

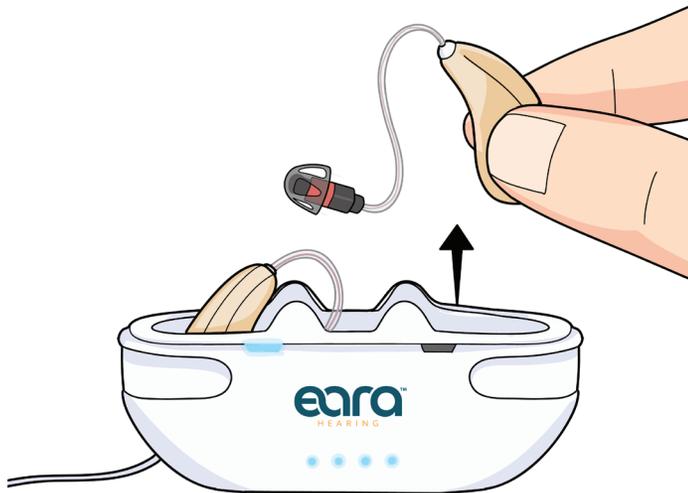

STEP 4: Pairing Hearing Aids to Bluetooth

After removing the hearing aids from the charger, either place them in your ears or set them on a clean, dry surface.

To pair with your smartphone:

1. Open Settings on your phone.
2. Turn on Bluetooth.
3. Scan for new devices.
4. Select Explore Li+ to pair.
5. Your hearing aids are now connected to your phone.

Explore Li+ Hearing Aids will now stream phone calls, music, and movies directly into your hearing aids.




For additional details, refer to page 8 of the User Manual.

STEP 5: How to Tell the Hearing Aids Apart

Each hearing aid is designated for a specific ear.

Left (L) Hearing aid= BLUE

Right(R) Hearing aid= RED

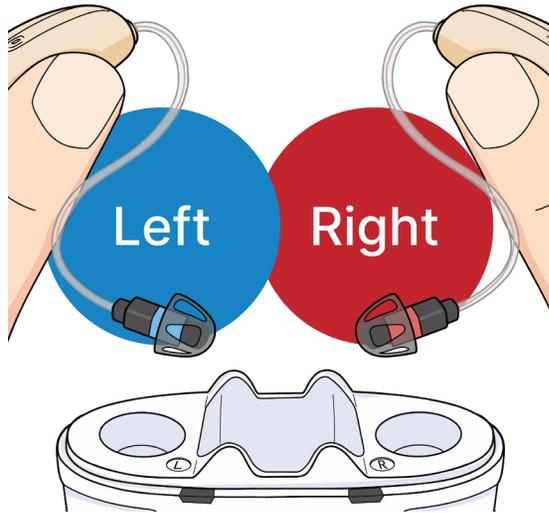

Blue beneath the dome indicates the Left Hearing Aid

Red beneath the dome indicates the Right Hearing Aid

STEP 6: Inserting Hearing Aids

1




Hook the hearing aid over the top of the ear so that the wire is close by the side of the head and the dome points into your ear canal.

2



Push the dome into your ear canal with your first finger and thumb.

3

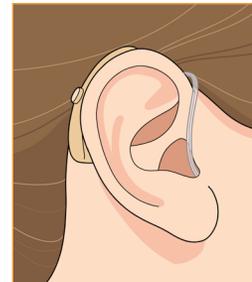

When the hearing aid is correctly inserted into your ear, you will not be able to see any part of the hearing aid in your ear.

STEP 7: Connect to the HA FIT

Now your new hearing aids are in your ears, let's connect your hearing aids to the HA FIT APP.

1. Open the **HA FIT APP** on your phone.
2. Select **Searching for Devices**.
3. Explore Li+ will appear for the right and left hearing aids.

STEP 8: Complete the Hearing Test

Find a quiet place to complete the hearing test to adjust your hearing aids to your listening needs.

1. When you are ready, begin by selecting **Hearing Evaluation**.
2. Select **Start**.
3. The test will begin. Try to listen for beeping sounds that you can just barely hear.
 - a. Select **Unheard**, if you do not hear the beeps.
 - **Tip:** Select **Unheard** only once, the sound will get automatically louder.
 - b. Select **Heard**, when you hear the soft beeping sounds.
 - Each time you select **Heard**, the test will continue to the next frequency tone.
 - Listen closely at each frequency, do you hear it?

Continue the steps above until you have reached the end of the hearing test.


For additional details, refer to page 18 of the User Manual.

STEP 9: Program the Hearing Aids

Congratulations!

You have successfully completed your hearing test.

1. Select **Auto-Optimize** to program your hearing aids to your hearing loss.
2. Once completed, select **Home** to go back to setting adjustments.

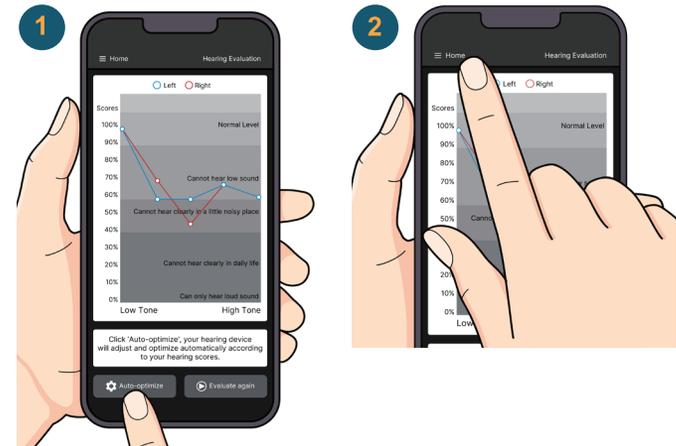



Quick Bluetooth Setup Guide

To get the most out of your new Explore Li+ hearing aids, please watch our short onboarding video before using your device.

It walks you through easy setup, Bluetooth tips, app pairing, and how to get the best sound experience right away.

Scan the QR Code below or by accessing:

<https://www.earahearing.com/pages/5-0-bluetooth-streaming>

